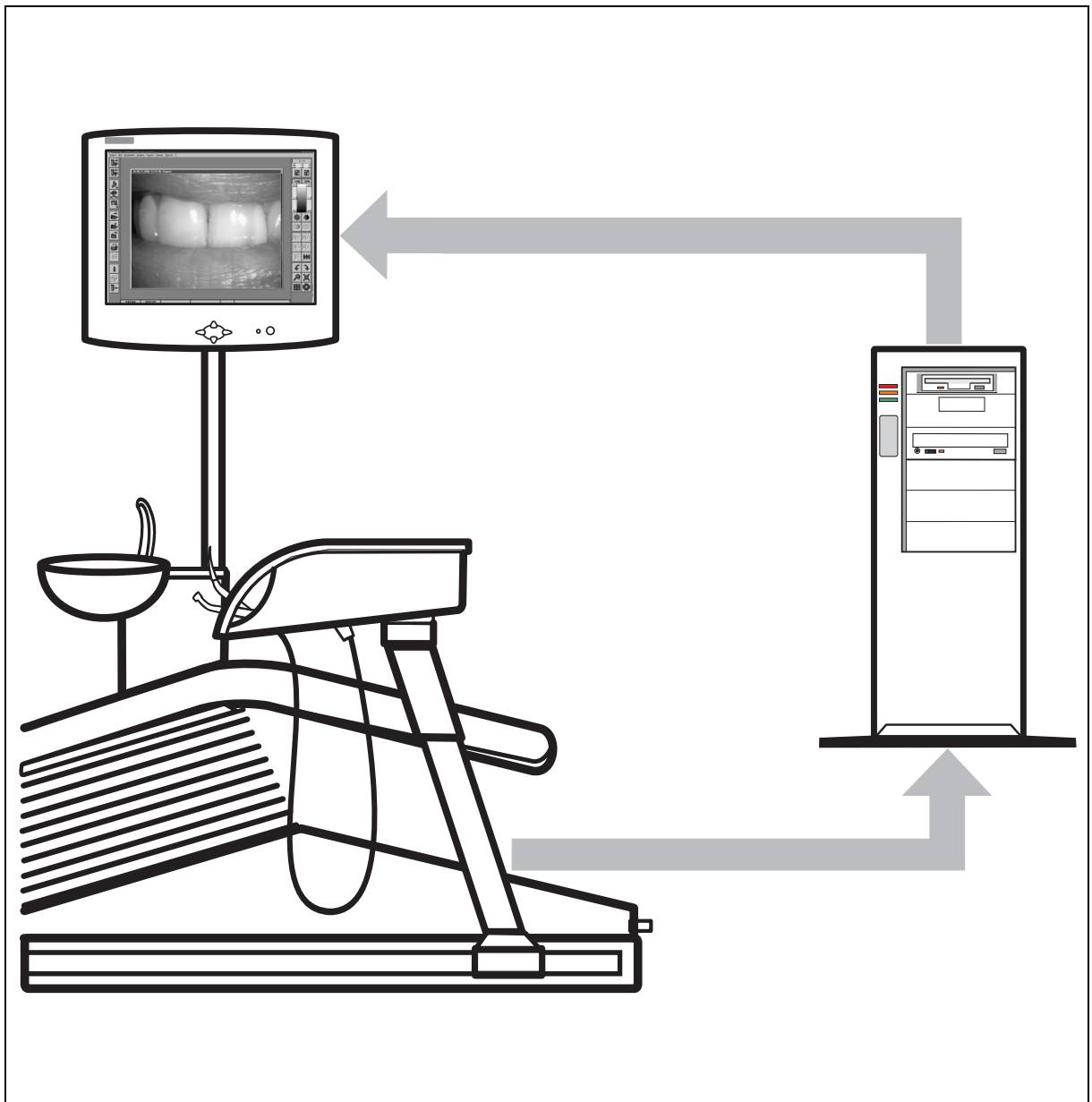
# SIVISION 1/2/3

Handbuch für den Anwender für PC-Software  
Operator's Manual for PC software  
Manuel de l'utilisateur pour PC-Software  
Manuel del usuario por software de PC

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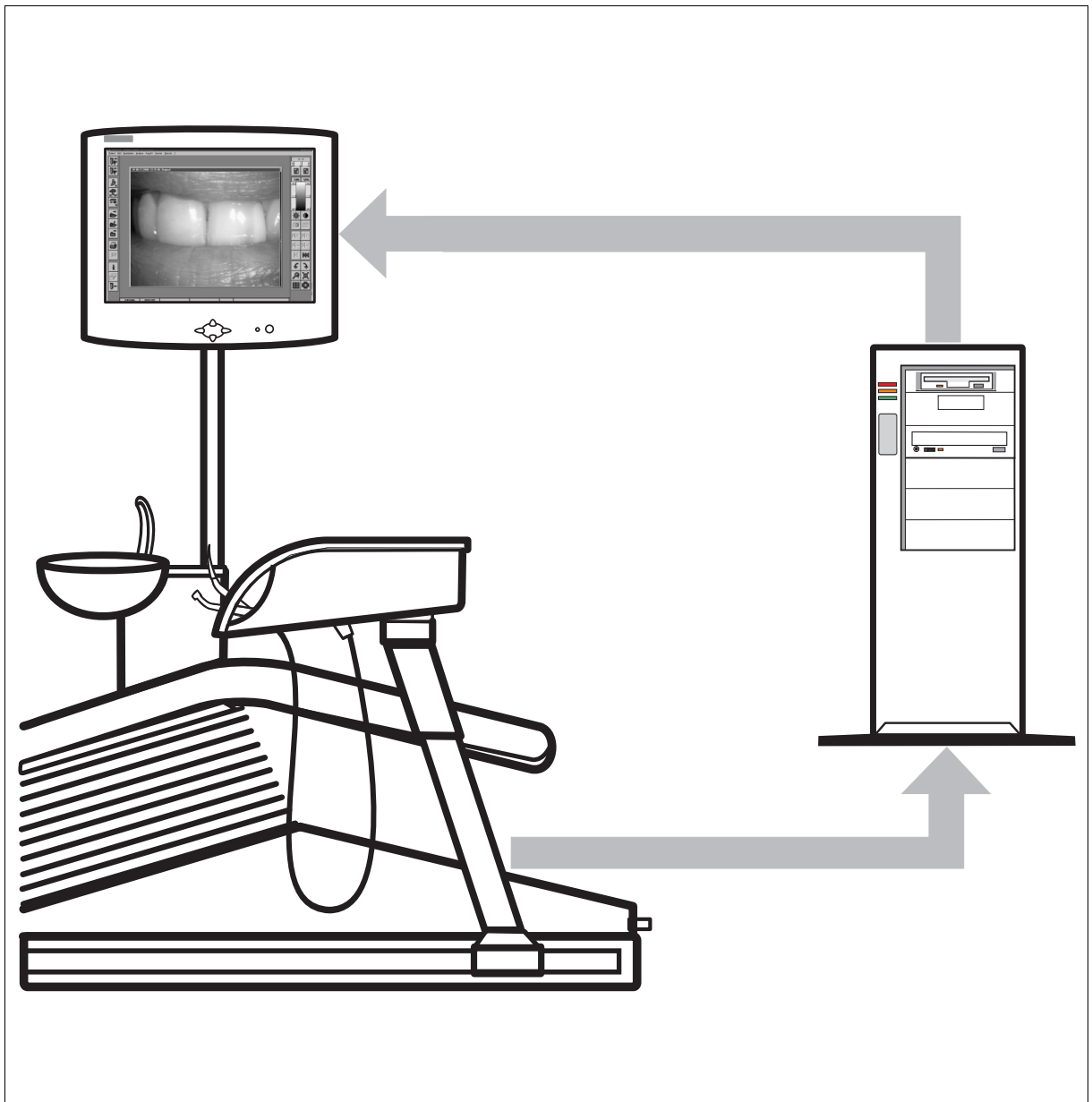
## 1

# General

The PC software for SIVISION makes it possible to control PC programs via the Sirona dental treatment units in connection with the SIVISION option and PC interfacing.

The PC functions can be operated via the user interface of the treatment unit (function keys) or the system foot switch.

This is realized by communication between the dental unit and the PC via the serial interface.

The function codes generated by specific control actions performed in the dental unit are allocated to the required PC functions in the SIUCOM program. Under certain circumstances it is therefore possible to start, terminate and even control PC applications. Furthermore, the display of a white or black image on the monitor of the dental unit has also been implemented.

When the PC software for SIVISION (SIUCOM) has been completely installed, the configuration of the PC control is preset depending on the dental unit selected for the control of SIDEXIS/VIDEXIS.

An overview of the factory settings is provided on a detachable cardboard sheet at the end of the manual.

The following chapters explain the system requirements, installation and configuration of the PC control as well as the control of outside applications.

# 2

## System requirements

### **Operating systems**

Windows 98 SE

Windows ME

Windows NT 4

Windows 2000

### **Dental treatment centers**

C1 and C1<sup>+</sup> with SIVISION 3

C2<sup>+</sup> with SIVISION 2

C3<sup>+</sup> with SIVISION 3

C4<sup>+</sup> with SIVISION 3

C6 with SIVISION 3

C8 with SIVISION 3

ProFeel with SIVISION 2 or 3

SIROCAM Box

SIVISION Compact

# 3

## Installation of the SIUCOM PC program

### 3.1 Installation procedure

#### ATTENTION!

If the user also plans to use **SIDEXIS/VIDEXIS<sup>1</sup>** software on the PC, this software must always be installed **before you install the SIUCOM** software.

#### Perform the installation as follows:

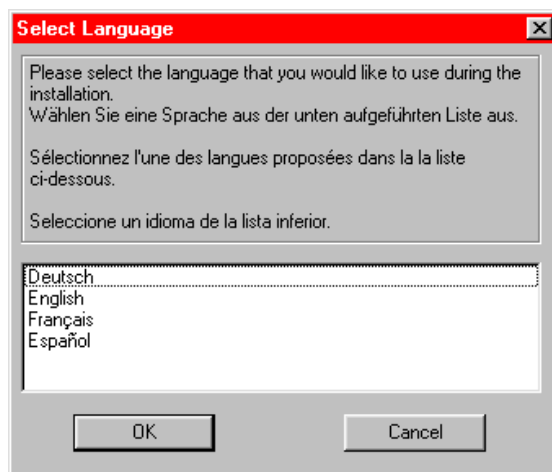
Please close all Windows programs currently running.

Insert the "**PC-Software SIVISION**" disk in the disk drive (DD).

Start **(DD):SIUCOM\Setup.exe**.

The installation program automatically detects the language version to be installed from the Windows country settings.

If the program currently set is not supported by SIUCOM, a language selection window appears on the screen. You can then use this window to select one of the languages supported by SIUCOM:


#### NOTE:

The language may be set only once during the installation procedure.

Mark the language you prefer and confirm your choice with **OK**.

1.SIDEXIS/VIDEXIS. SIRONA PC Software for the acquisition, analysis, processing and archiving of dental images with a patient database.

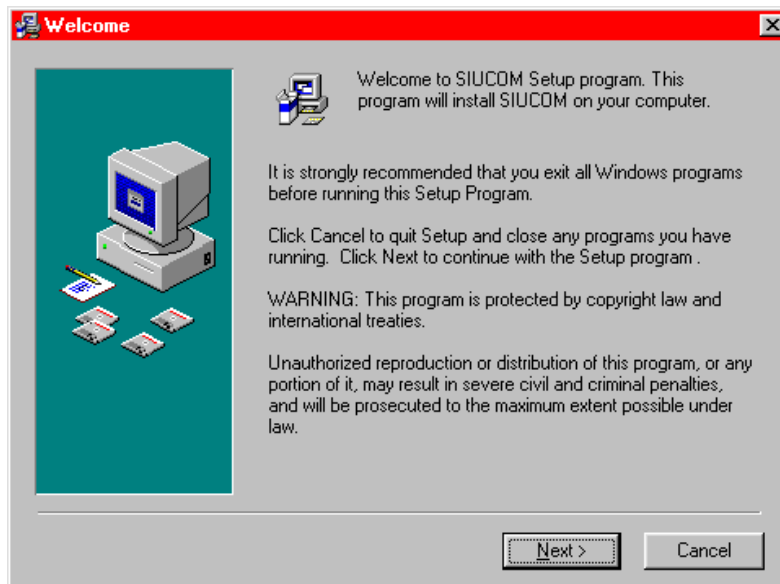
The Startup window appears:

Proceed as instructed in this window.

Click on "**Next**".

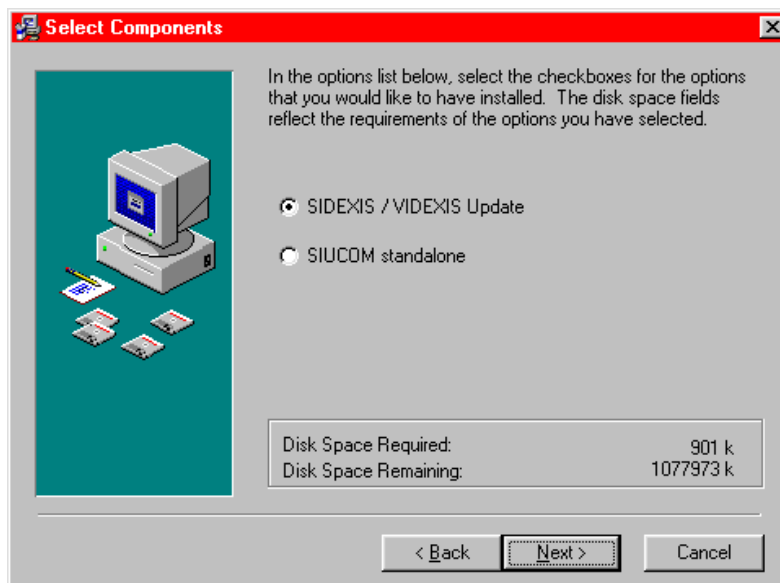The "**Select components**" window appears:

If you work with SIDEKIS/VIDEKIS, select "**SIDEKIS/VIDEKIS Update**".

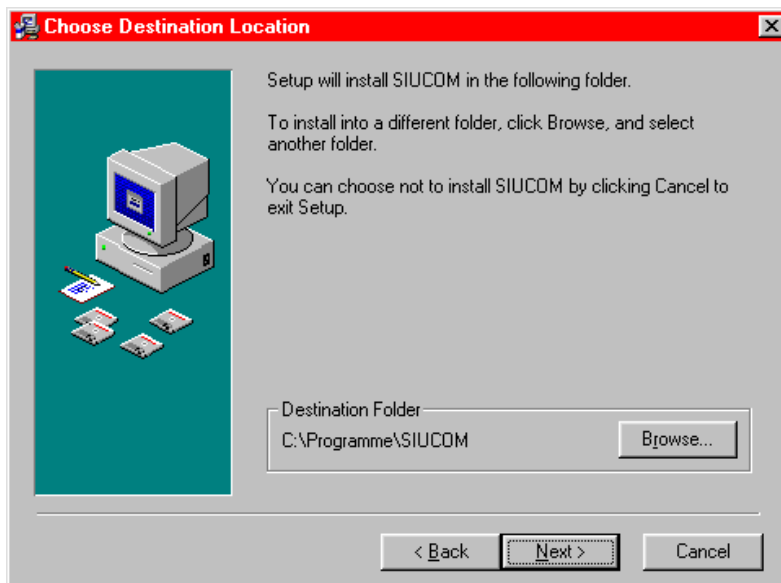
SIDEKIS/VIDEKIS must already be installed, the new version is installed in the SIDEKIS/VIDEKIS directory.

If a configuration without SIDEKIS/VIDEKIS is planned, select "**SIUCOM standalone**".

Following this selection, click on "**Next**".


If you have selected "**SIUCOM standalone**", you now have the option of selecting the target path where **SIUCOM** should be installed.

If you want to install **SIUCOM** in any directory other than the one recommended, you may also click on the "**Browse...**" button and specify a different directory.


Click on "**Next**".

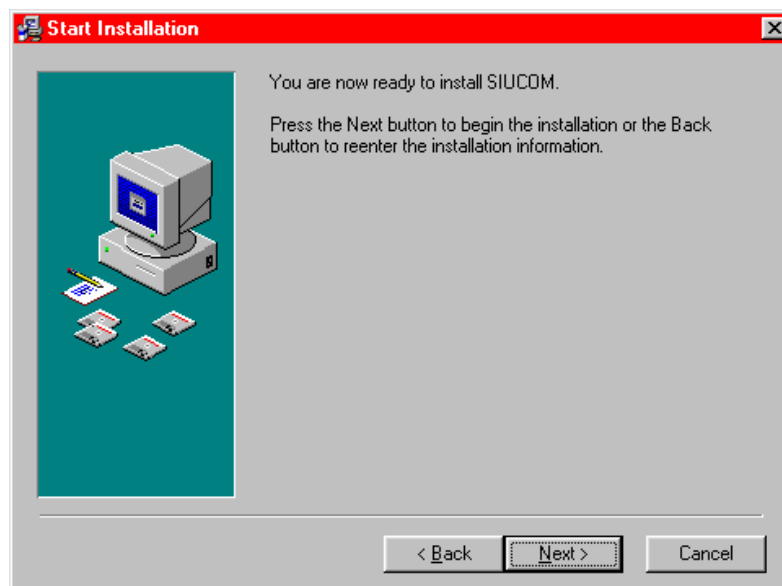The Setup window for beginning the installation appears:

Begin the installation procedure by clicking on "**Next**".

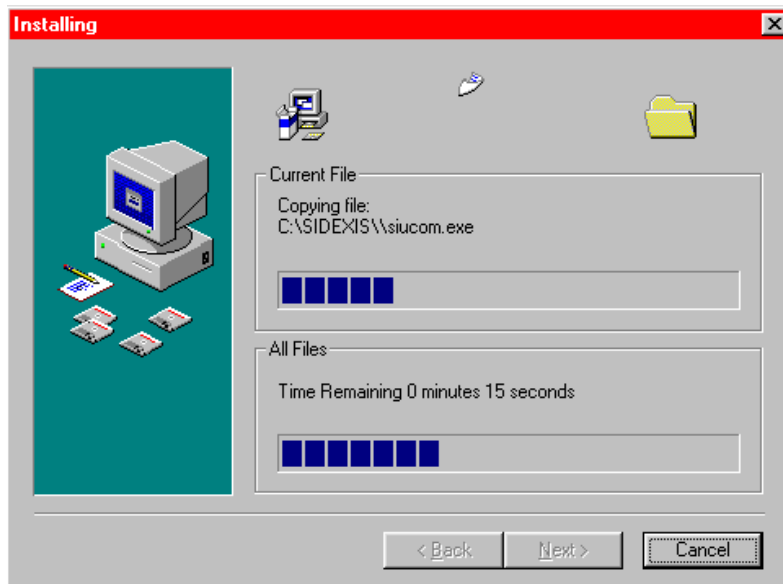
The Setup window appears:

The **PC software for SIVISION** is then installed; no entries are required.

The window automatically closes again.

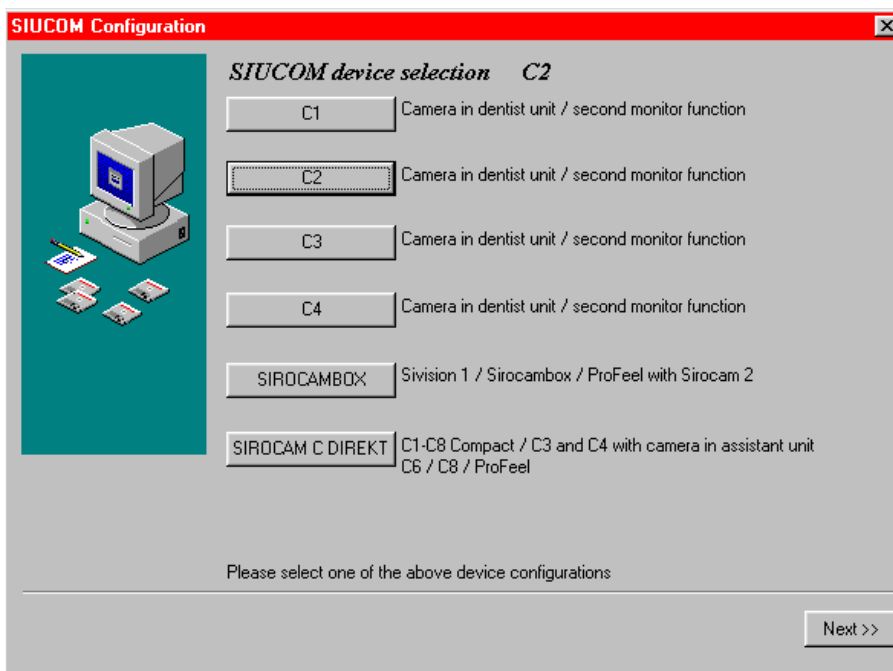A new window for device selection then appears:

Here you can select the device configuration corresponding to the camera integrated by clicking it on. The default selection is SIROCAM Box. In the above illustration we have selected C2.


The device configuration currently selected is then displayed in large print in the heading.

**NOTE:**

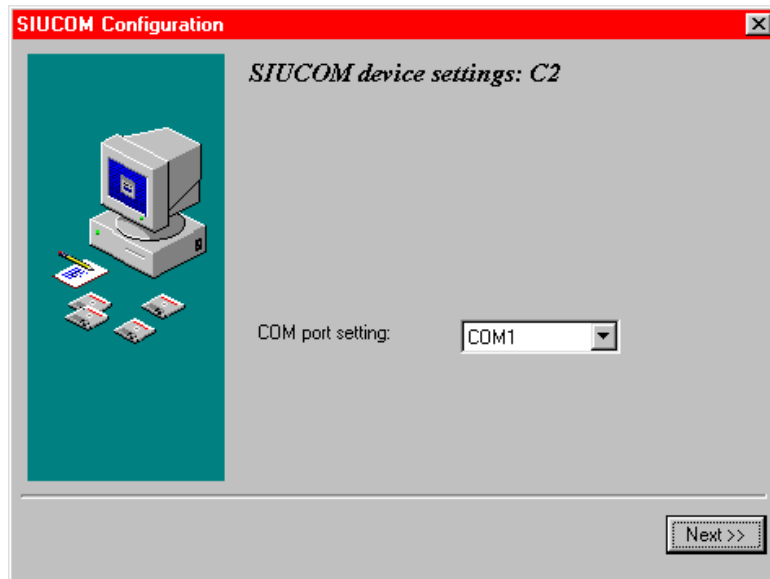*You can also change the device selection any time following installation.*

Following this selection, click on "**Next**".

A "SIUCOM configuration" window appears:

Select the **COM port** to which the selected device is connected.

**NOTE:**

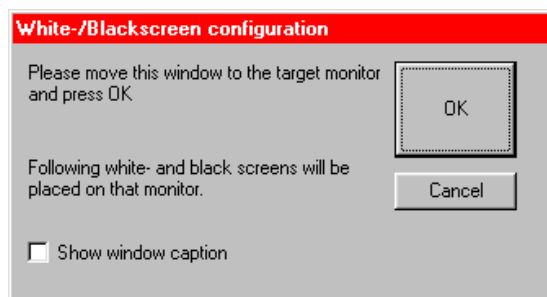
*This setting can also be changed following installation.*

Click on "Next".

The setting window for the white/black image then appears:

**NOTE:**

*This setting must be made even if the device previously selected does not offer the possibility of controlling a white/black image:*


If second monitor or multimonitor operation is planned, click on the title bar of this window and, holding the mouse button down, drag it to the monitor where the white/black image should appear in the future.

Then you must click on the "OK" button there.

**NOTE:**

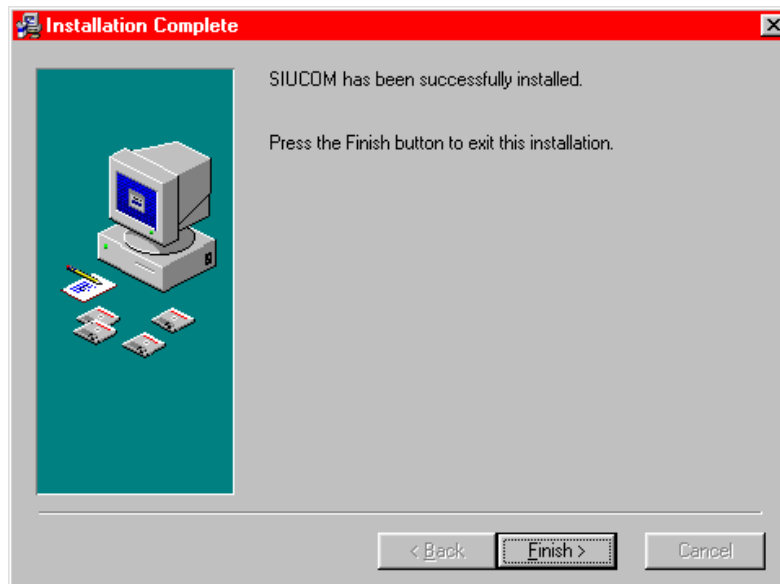*This setting can still be changed following installation.*

The window for quitting Setup then appears:

Click on "**Finish**".

**The installation of SIUCOM is now finished.**

Remove the installation disk from the disk drive.

## 3.2 Function check

The **SIUCOM** program is automatically started by **SIDEXIS/VIDEXIS**. In the **standalone configuration** the **SIUCOM** program can be called up from the Start menu by selecting **Start, Programs, SIUCOM, SIUCOM**.

### • Configuration with **SIDEXIS/VIDEXIS**:

Start **SIDEXIS/VIDEXIS**.

Remove the video camera. A **LIVE IMAGE** must then appear in **SIDEXIS/VIDEXIS**.

#### **NOTE:**

*For dental units with SIVISION 3 and a camera in the dentist element, please make sure that "**PC-Mode**" is set on the display of the dentist element (see the operating instructions for the unit concerned).*

→ If **no LIVE IMAGE** appears, check whether the **SIUCOM icon** is displayed at the bottom right of the screen.

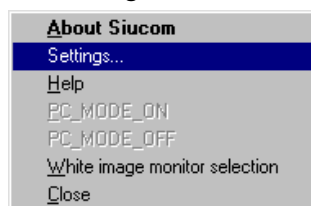
If this icon is missing, the configuration with **SIUCOM** must be released in **SIDEXIS/VIDEXIS**.


For details, please refer to the **SIDEXIS/VIDEXIS** installation instructions.

- If the **SIUCOM icon** is displayed at the bottom right, however **no LIVE IMAGE** appears and no other reaction occurs, click on the **"Video recording"** button in SIDEXIS/VIDEXIS. If a live image now opens, there is a fault in the serial data transmission from the dental unit to SIUCOM. Check whether the correct COM port is set.

Click on the **SIUCOM icon** with the **right mouse button** to open the SIUCOM menu. Select the **"Settings"** command there.

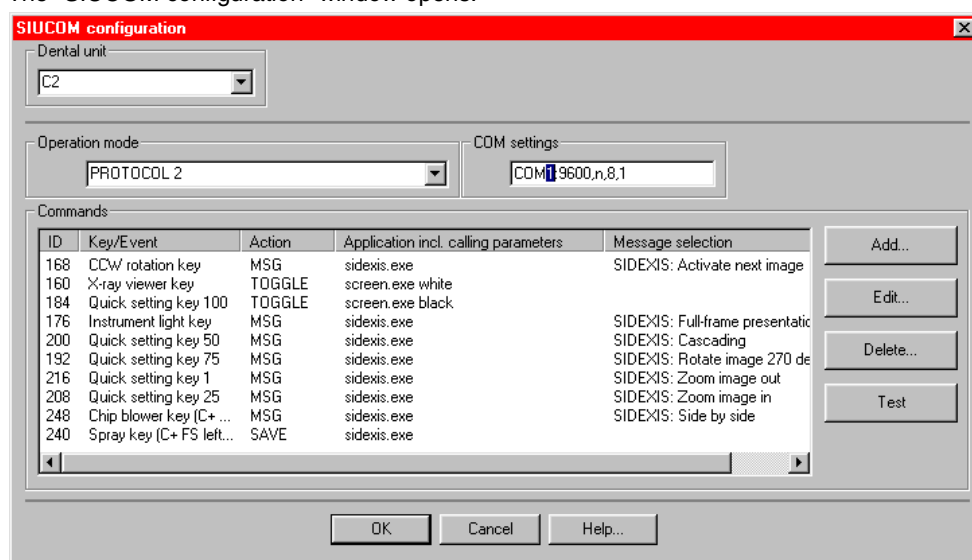The "SIUCOM configuration" window opens:

Please check the **COM port** setting specified here under **"COM settings"** by comparing it with the serial interface designated on the PC and correct by overwriting it if necessary. If the COM port setting is correct, check the serial lead connecting the treatment unit to the PC.

- If a black window appears instead of a LIVE IMAGE window shortly after the camera is removed, the S-video signal is missing. Please check the plug connections, the camera lead and the S-video lead to the PC.

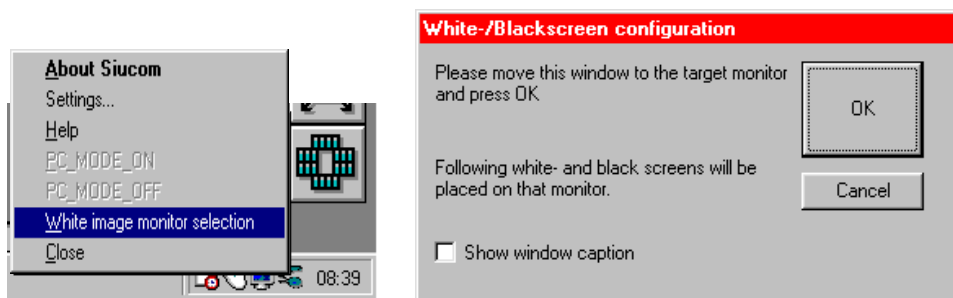
• **Test of white/black image for the following configurations in the Multimonitor mode:**

- C1<sup>+</sup> with second monitor
- C2<sup>+</sup>
- SIROCAM C Direct

Test whether the **white/black images** appear on the correct monitor. This is best done by switching them on with the appropriate white or black image function key as described in the operating instructions for the treatment center concerned.

- If a white/black image appears on the wrong monitor, click on the **SIUCOM icon** with the **right** mouse button to open the SIUCOM menu. There you select the command "**White image monitor selection**".

The setting window for the white/black image then appears:

Click on the title bar of this window and, holding the mouse button down, drag it to the monitor where the white/black image should appear.

Then you must click on the "**OK**" button there.

- If the white/black image appears on both monitors, place a check mark in "**Display window title bar**" in the upper setting box.

The white/black image appears on the monitor where the setting window is located after you click on the **OK** button.


## 4
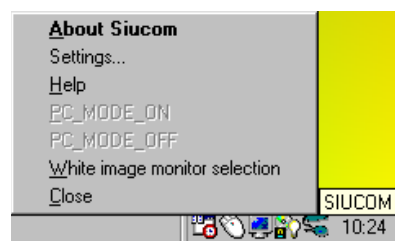
# Configuration

## 4.1 SIUCOM pop-up menu

When SIUCOM is started (manually via **Start, Programs, SIUCOM, SIUCOM** or automatically by SIDEXIS/VIDEXIS), the **SIUCOM** icon appears in the lower right corner of the monitor. Click on this icon with the **right** mouse button. The **SIUCOM menu** then appears:

### Concerning Siucom:

A brief info about the SW Version, is then displayed.

### Settings

opens the "SIUCOM configuration" window. Here you have the possibility of changing various settings and configurations for SIUCOM (see Section 4.2).

### Help

starts a brief online help routine.

**PC\_MODE\_ON / PC\_MODE\_OFF** These two commands can be activated only if the "**SIROCAM C Direct**" is selected. Using them you can set the operating mode of the SIROCAM C:

Using PC\_MODE\_ON you can switch the SIROCAM C video camera to the PC mode and PC\_MODE\_OFF to deactivate the PC mode (also refer to operating instructions of dental unit concerned).

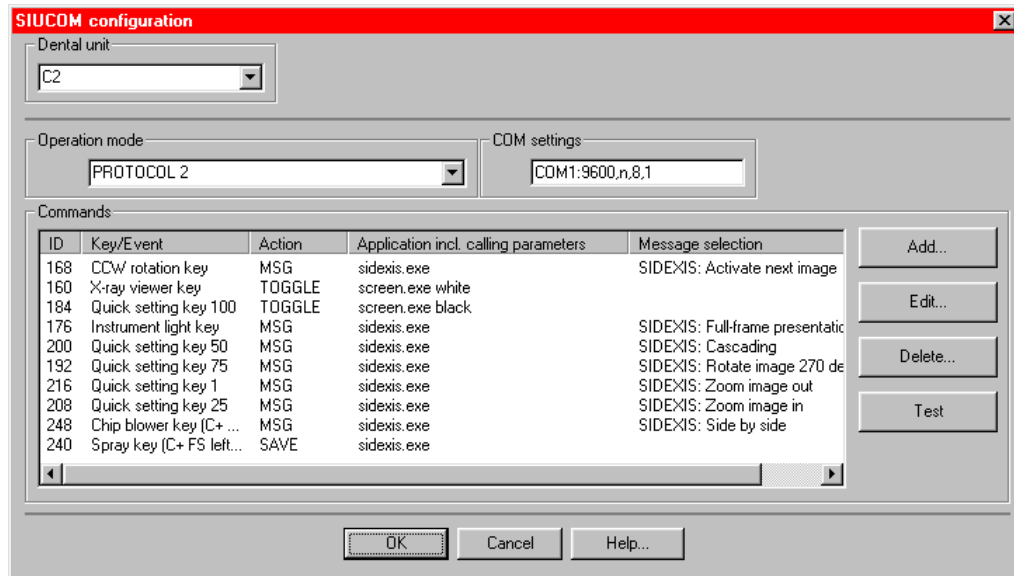
**White image monitor selection** opens the window used to select the monitor for white/black image display (see Section 3.2 "Function check").

### Close

You can use this command to quit SIUCOM. However, first a window will appear and ask if you really want to quit SIUCOM. Closing is possible only if SIUCOM standalone is running (without SIDEXIS/VIDEXIS). If SIUCOM was installed with SIDEXIS/VIDEXIS, it will always automatically be started and terminated along with SIDEXIS/VIDEXIS, i.e. it can not be closed manually as long as SIDEXIS/VIDEXIS is running.

## 4.2 SIUCOM Configuration

The "SIUCOM configuration" window shows the factory setting for C2<sup>+</sup>:

- Dental unit** – Picklist for treatment unit and/or device configuration.
  - C1** – camera in dentist element / second monitor function
  - C2** – camera in dentist element / second monitor function
  - C3** – camera in dentist element / second monitor function
  - C4** – camera in dentist element / second monitor function
  - SIROCAM C Direct** – C1-C8 Compact / C3 and C4 with camera in assistant element / C6 / C8 / ProFeel.
  - SIROCAM Box** – SIVISION 1 / SIROCAM Box / ProFeel with SIROCAM 2
- Operating mode** – Picklist for protocol variant
 

*Protocol 1:*  
 SIDEXIS/VIDEXIS still image is saved by actuating the foot switch pedal.  
 For use with the following dental units:  
 SIROCAM C Direct (C1 – C8 Compact / C3<sup>+</sup> and C4<sup>+</sup> with camera in assistant element / C6 / C8 / ProFeel), SIROCAM Box, SIVISION 1, ProFeel with SIROCAM 2 in connection with SIDEXIS/VIDEXIS.

*Protocol 2:*  
 SIDEXIS/VIDEXIS still image is saved with a separate function key.  
 Recommended for the following dental units:  
 C1<sup>+</sup> / C2<sup>+</sup> / C3<sup>+</sup> / C4<sup>+</sup> with camera in dentist element.

*Protocol 3:*  
 Enables the operation of a camera not integrated in the dentist element in connection with SIDEXIS/VIDEXIS for the following dental units:  
 C1<sup>+</sup> / C2<sup>+</sup> / C3<sup>+</sup> / C4<sup>+</sup>  
 (Example; see Section 6.1).

*Protocol 4:*  
 This protocol is intended for future extensions or upgrades in connection with other PC applications.

**3. COM settings:**

COM1 – Designation for serial interface of PC (depending on port used)

9600 – Baud rate, preset depending on the treatment center

n,8,1 – Additional information (do not overwrite)

**4. Commands – List of configurations:**

**ID, Key/Event** – Identity number and name of set function keys, foot switch actions and camera status.

**Action** – Type of action - Preselection of function keys and foot switch functions:

**TOGGLE** – Each time you press this key the linked application will be started or closed. An application which is already running in the background mode will be brought to the foreground.

**ACTIVATE** – The linked application will be started and/or brought to the foreground when the key is pressed.

**STOP** The linked application will be closed when this key is pressed.

**SAVE** – Special command to save video still images in SIDEXIS/VIDEXIS according to date and time (no file name has to be entered).

**MSG** – "Message" enables you to control various different functions in application programs.

**Application incl. calling parameters** – The given PC program which is to be operated is displayed with this command.

**Message selection** – Used to display which function from the preselected PC program is being controlled.

**Buttons in the "SIUCOM configuration" window:****Add...**

The "Command configuration" window (see Section 4.3) opens to enable the entry of new settings.

**Edit...**

The "Command configuration" window (see Section 4.3) is opened with the marked command. Here you can make changes in the selected command.

**Delete...**

The marked command is deleted.

**Test**

Tests the functioning of the marked command without requiring any control actions on the dental unit (however, when testing the TOGGLE action, no closing is possible due to the nature of the function).

Please note that it is not possible to operate the PC from the dental unit as long as the "SIUCOM configuration" window remains open.

**OK**

The "SIUCOM configuration" window is closed and all of the changes which have been made are saved.

**Cancel, ☒ Close**

The "SIUCOM configuration" window is closed without saving the changes.

**Help...**

A short online help message is displayed.

### 4.3 Command configuration

After "Add..." the "Command configuration" window appears without presettings.

After "Edit..." the entries of the previously selected command are preset.

#### 1 – Key selection

If you call in the list with "Key/Event", all available function keys, foot switch functions and events (e.g. camera removed, camera deposited, etc.) will be displayed for individual selection.

#### NOTE:

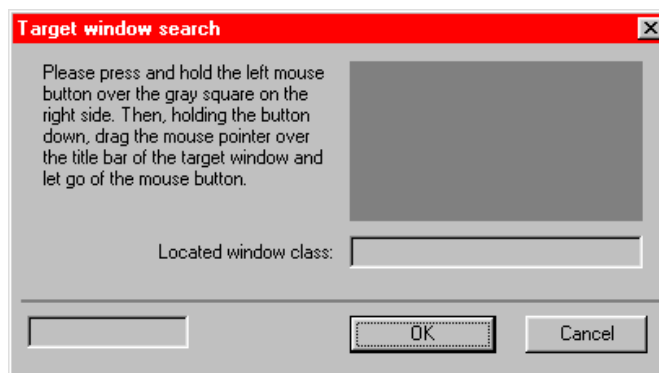
Please note that in order to use the **Hash mark key deactivated** key available on some dental units you must first set the **# function** on the dental unit to **Switch**.

#### 2 – Key action

Select the action required under "Action".

- **TOGGLE, ACTIVATE, STOP:** If you have selected **TOGGLE, ACTIVATE** or **STOP**, you must then specify a program file under **Application incl. calling parameters**. You can select this file either by manually entering a path or by clicking on the "Search..." button. The "Search..." button opens a window in which you can select a program file for transfer to the **Application incl. calling parameters** field. Also enter the calling parameters if required.

In order to determine the **window class** the selected application must be opened and at least part of its title bar must be visible.  
Click on the **Determine...** button .  
The **Target window search** window then opens.

Click on the gray field in the window with the left mouse button and hold the button down. Then, holding the button down, drag the mouse pointer over the title bar of the opened application (\*.exe). The window class will then be found and displayed.

Click on **OK** to accept the window class.

For examples, refer to Sections 6.2 and 6.3.

- **SAVE (Save command for SIDEXIS/VIDEXIS):** For **SAVE** under Application incl. calling parameters you must enter the calling parameter "**sidexis.exe**" and under Window class "**MdiXFrameClass**".

→ **MSG:** If you select **MSG** from the Key action list, a **Message selection line** will be added to **field 2 Key action** .

**Command configuration**

**1** Key selection  
Please select the button/key required

Dental unit: C2

Key/event: Quick setting key 50

**2** Key action  
Please specify the behavior required by defining a corresponding action

ID: 200

Action: MSG

Message selection: SIDEXIS: Close active image

Message ID: 715

Application incl. calling parameters: sidexis.exe Find...

Window class: MdiXFrameClass Determine...

The window class can be determined interactively by pressing the Determine buttons. However, the required application must be started beforehand!

OK Cancel

Select the **SIDEXIS/VIDEXIS function** which you require for the preselected treatment center function key from the list displayed here. The remaining entries **Application incl. calling parameters** and **Window class** then appear automatically.

## 5

# Operation

No PC-controlled operation of SIUCOM is planned.

**Control takes place at the dental unit.**

The prerequisite for controlling the PC from the dental unit is that the SIUCOM program has been started on the PC.

- SIUCOM is **automatically** started and terminated in connection with **SIDEXIS/VIDEXIS**.
- **SIUCOM standalone** (without SIDEXIS/VIDEXIS) must be started manually (via **Start, Programs, SIUCOM, SIUCOM**) or via a link in the "Autostart" folder.

To close SIUCOM in the standalone mode, you must click on the SIUCOM icon with the right mouse button and then select the **Close** option.

When SIUCOM is started, the SIUCOM icon appears in the task bar (usually in the bottom corner right of the screen).

The allocation of dental unit actions to PC functions can largely be configured to meet individual demands (see Chapter 4).

The following information therefore generally refers only to the factory settings.

A live image is automatically opened in the SIDEXIS/VIDEXIS program on the PC when the camera is removed at the dental unit and closed again when the camera is deposited.

Actuation of the foot switch changes over between the live and still image modes.

In C1<sup>+</sup>, C2<sup>+</sup>, C3<sup>+</sup> and C4<sup>+</sup> dental units with the camera in the dentist element, the current still image is saved with the Spray key (or, when the camera is removed, with the left pedal of the C<sup>+</sup> foot switch). In all other dental units, the still image is saved by actuating the foot switch longer than one second.

It is advisable to preassign SIDEXIS/VIDEXIS functions, white image or black image to several of the function keys available depending on the dental unit selected.

These factory settings are listed on the detachable operating cards provided at the end of the manuals for the different Sirona dental units.

You can enter your own configuration on the reverse side of the operating cards.

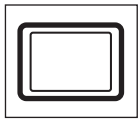
**Explanation of the symbols which can be used on the operating cards or for your own configuration:**

Not assigned ex works

No PC control function possible


White image

White image / black image


In addition to the possibility of controlling the white/black image from the dental unit, you can also **close** it with the **Esc key** or the **right mouse button**.

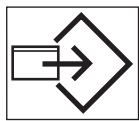
Black image

Saving images

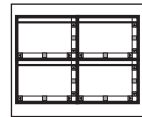Side by side

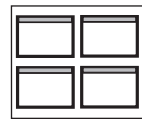
Window Overview


(not available before SIDEXIS/  
VIDEXIS version 5.5)

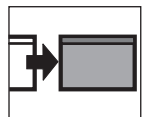
Cascading

Activate next image

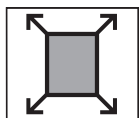Full-frame presentation / Recover

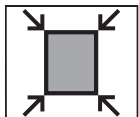
Restore normal size

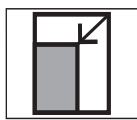
Zoom image in


Zoom image out

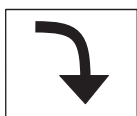
Rotate image 90° clockwise

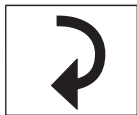
Rotate image 90° counterclockwise


Rotate image 180°

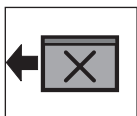
Close active image

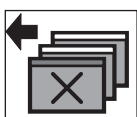Close all images

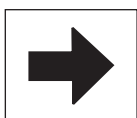Next logical step forward

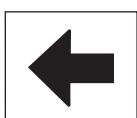Next logical step backward

Change between live and still image

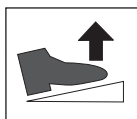
Camera on

Camera off

Foot switch pedal activated

Foot switch pedal deactivated

Short click of foot switch pedal

## 6

# Examples

## 6.1 Sample configuration for Protocol 3:

Based on the standard configuration, operation involving a camera not integrated in the dentist element can be configured as follows in C1<sup>+</sup>, C2<sup>+</sup>, C3<sup>+</sup> and C4<sup>+</sup> dental units:

**SIDEXIS/VIDEXIS** must be opened.

Click on **SIUCOM** icon (bottom right corner of screen) with right mouse button and select the **"Settings..."** menu option .

The **"SIUCOM configuration" window** opens.

Set **Protocol 3** as the operating mode.

### For C1<sup>+</sup>:

Click on the **"Add..."** button.

The **"Command configuration" window** opens.

Make the following entries:

Key/Event: **"CAM\_CONTROL: Foot switch pedal activated"**

Action: **"MSG"**

Message selection: **"SIDEXIS\_CONTROL: short click of foot switch pedal"**

Quit "Command configuration" window with **"OK"**.

Click on the **"Add..."** button.

The **"Command configuration" window** opens.

Make the following entries:

Key/Event: **"Key F7"**

Action: **"MSG"**

Message selection: **"SIDEXIS\_CONTROL: Camera on"**

Quit "Command configuration" window with **"OK"**.

In the "Commands" list double-click the line **„200 Key F8 ...“** .

The **"Command configuration" window** opens.

In the Message selection field: Select the list entry **"SIDEXIS\_CONTROL: Camera off"**.

Quit "Command configuration" window with **"OK"**.

**For C2+, C3+ and C4+:**

In the "Commands" list double-click the line **"216 quick setting key 1 ..."** .

The **"Command configuration" window** opens.

In the Message selection field: Select the list entry **"SIDEXIS\_CONTROL: Camera on"**.

Quit "Command configuration" window with **"OK"**.

**For C2+:**

In the "Commands" list double-click the line **"208 quick setting key 25 ..."** .

**For C3+ and C4+:**

In the "Commands" list double-click the line **"200 quick setting key 50 ..."** .

The **"Command configuration" window** opens.

In the Message selection field: Select the list entry **"SIDEXIS\_CONTROL: Camera off"**.

Quit "Command configuration" window with **"OK"**.

Click on the **"Add..."** button.

The **"Command configuration" window** opens.

Make the following entries:

Key/Event: **"Forward cursor from instrument keypad"**

Action: **"MSG"**

Message selection: **"SIDEXIS\_CONTROL: short click of foot switch pedal"**

Quit "Command configuration" window with **"OK"**.

Quit "SIUCOM configuration" window with **"OK"**.

**Operation is then as follows for removing/depositing the camera and still/live image selection (with video signal present):**

**For C1+:**

To **remove** the camera, press **key F7**.

To **change** between **still and live image** step on the foot switch pedal.

To **deposit** the camera, press **key F8**.

**For C2+, C3+ and C4+:**

To **remove** the camera, press **quick setting key 1**.

To **change** between **still and live image**, actuate the **cursor point** via the instrument keypad.

To **deposit** the camera, press **quick setting key 25 (C2+)**

or **50 (C3+ and C4+)** .

## 6.2 Example of command configuration with **ACTIVATE**, **STOP** or **TOGGLE**:

### **NOTE:**

*The actual path information and file names may vary from this example depending on the operating system and type of installation involved.*

Assuming that you would like to view a video file on the PC from the dental unit which you had previously saved on the PC:

Open the "**Command configuration**" window as described in Chapter 4.

Select the key with which you would like to start or close the video, e.g. the calling key.

Select the **TOGGLE** action (starts and stops an application).<sup>1</sup>

Click on the "**Search...**" button go to the directory C:\Programs\ Windows Media Player and select the file mplayer2.exe. Quit the window by clicking on the **Open** button.

The field **Application incl. calling parameters** now contains the program file just selected.

Add a blank, a quotation mark, and the name of the video file incl. its path and a closing quotation mark to the end of the line. In our example the contents of the **Application incl. calling parameters** field now have the following appearance:

*C:\Programs\Windows Media Player\mplayer2.exe "C:\My Documents\example.mpe"*

Now start the Windows Media Player via Start, Programs, Accessories, Entertainment Media.

Bring the "**Command configuration**" window back to the foreground and click on the "**Determine...**" button there.

Determine the window class of the Windows Media Player as described in the window.

Close all SIUCOM configuration windows by clicking on their respective "**OK**" buttons.

Close Windows Media Player too.

### **To control the video from the dental unit, proceed as follows:**

Press the calling key: The Windows Media Play is opened and the selected video starts running.

Press on the calling key again to close the Windows Media Player.

<sup>1</sup>. Instead of the TOGGLE action, you could also select the ACTIVATE action. Then you require an additional function key to which you must have assigned the STOP action. Or alternatively, just close the application directly from the PC via the mouse or keyboard.

---

### 6.3 White image / black image:

Open the "**Command configuration**" window as described in Chapter 4.

Select the key with which you would like to display the white or black image.

Select an action (TOGGLE, ACTIVATE or STOP), we recommend selecting **TOGGLE** for this purpose.

- For the white image under **Application incl. calling parameters** you must enter "**screen.exe white**" and "**SIRONA WHITE IMAGE**" for the window class.
- For the black image, under **Application incl. calling parameters** you must enter "**screen.exe black**" and under image class "**SIRONA BLACK IMAGE**".

---

We reserve the right to make any alterations which may be due to technical improvements.

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© Sirona Dental Systems GmbH 2001
## **Sirona Dental Systems GmbH**
Sirona USA, LLC
Ltd./LTE